

# SILICONE LUBRICANT FOR PROSTHETIC EYE WEARERS.

Ocu-Sil™ is an optimal viscosity liquid silicone oil used as a lubricant for ocular prosthesis. This silicone lubricant relieves irritation and discomfort for the ocular prosthesis wearer. Ocu-Sil also aids in the control of excessive external discharge from the socket of the prosthesis wearer.

## Instructions:

1. Apply 1-2 drops of Ocu-Sil 3-4 times daily without removing the artificial eye. Blinking will help to spread the lubricant evenly. If excessive lubricant is present remove with cleansing tissue until desired lubrication is reached.
  2. Cleaning topical surface of eye. Dampen a cotton-tipped applicator, without removing the eye; gently pull down on the lower lid and remove any accumulated discharge. Do this as often as is necessary to maintain appearance.
  3. Leave the eye in place as long as it is comfortable.  
Remove it for either:
    - A. excessive external discharge if noticed soon after cleansing with the applicator.
    - B. Unusual irritation that is not relieved with the usual addition of Ocu-Sil.
1. Eye Removal:
    - A. Wash and rinse hands thoroughly.
    - B. Remove the eye as usual. Wash and rinse eye thoroughly.
    - C. Dry the eye with a cleansing tissue, then apply Ocu-Sil to the front and back surfaces.
    - D. Replace the eye and add one additional drop externally.

Use only as intended.

## Warning:

For external use only. Do not use if solution changes color or becomes cloudy.

## When using this product:

- To avoid contamination, do not touch tip of container to any surface.
- Do not use if seal has been broken.
- Only use for prosthetic eye. Do not put in seeing eye.

## Consult a physician if:

- You experience eye pain.
- Continued Irritation occurs.
- Condition worsens or persists for more than 72 hours.

## Keep out of reach of children:

If swallowed contact a poison control center immediately.

Ocu-Sil is a optimal viscosity dimethylpolysiloxane silicone and is fast becoming preferred by ocular prosthesis wearers for irritation relief, discharge, and crusting.



(15cc)

Order Online:

[www.ocu-sil.com](http://www.ocu-sil.com)

Order by Phone:

1-888-OCU-SIL5

(1-888-628-7455)

Distributed by:

**MSM Industries, LLC**

4 NE 10th St. PMB 401

Oklahoma City, OK 73104-1402

USA